Venus®

| | | |
|----|----------------------------|----|
| Ⓓ | Gebrauchsanweisung | 3 |
| ⒼⒷ | Instructions for use | 9 |
| Ⓕ | Mode d'emploi | 15 |
| Ⓔ | Instrucciones de uso | 20 |
| Ⓘ | Istruzioni per l'uso | 26 |
| Ⓟ | Instruções de uso | 31 |
| ⓃⓁ | Gebruiksaanwijzing | 37 |
| Ⓢ | Bruksanvisning | 43 |
| ⒹⓀ | Brugsvejledning | 48 |
| Ⓝ | Bruksanvisning | 53 |
| ⒻⒾ | Käyttöohje | 58 |
| ⒼⓇ | Οδηγίες Χρήσης | 63 |

Heraeus

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Universal light curing composite, radio-opaque

VENUS is a lightcuring, radiopaque composite. It is indicated for adhesively bonded, tooth colored restorations. It is suitable for anterior and posterior applications. Due to the microglass filler system, a beautiful esthetic result and a durable, high lustre polish is easily realized.

Composition:

VENUS is based on a BIS-GMA matrix and contains 61% filler (by volume), which is:

- Barium Aluminium Fluoride glass (\varnothing 0.7 μm ; max. < 2 μm)
- Highly dispersive Silicon Dioxide (\varnothing 0.04 μm)

Clinical Indications:

- Direct restoration of Class I-V cavities (according to G.V. Black)
- Direct composite veneers
- Shape corrections of teeth (ie. diastemas, congenital defects in teeth, etc.)
- Temporary splinting of teeth loosened by trauma or periodontal disease
- Indirect restorations (inlays, veneers)
- Restoration of primary teeth
- Core build-up
- Temporary repairs of porcelain, composite (in combination with an adequate repair-system)

Clinical treatment protocol:

Select an appropriate shade or shade combination from the **VENUS** Shade guide prior to placing the rubber dam. This is important because teeth will significantly change color when desiccated. If the tooth surface being treated is not to be instrumented by a diamond stone, carbide bur, drill or other abrasive instrument, the surface should be cleaned with pumice or a fluoride free polishing paste before restoration is placed. A non-sticking plastic, Teflon coated or gold plated placement instrument (such as the Heraeus Kulzer Plasmacoat Instrument) is recommended for use in clinically placing, sculpting and shaping the material.

1. For ideal results, the use of an Ivory rubber dam (or similar) is recommended to control contamination from moisture, blood and saliva.
2. Prepare the cavity according to generally accepted rules for the clinical composite application. Then clean and dry the cavity.
3. Prior to application of **VENUS** restorative material to the cavity preparation, pre-treatment of the tooth structure with an adequate enamel/dentin adhesive is mandatory (e.g. GLUMA® Comfort Bond + Desensitizer, GLUMA® Comfort Bond, GLUMA® Solid Bond by Heraeus Kulzer). Once treated with the adhesive, avoid contamination with saliva or blood.
4. Apply **VENUS** Restorative material in thin layers (max. 2 mm) and adapt carefully to the tooth surface or cavity walls. In case of severe discoloration of the underlying tooth or surface being restored, it may be necessary to mask this by using a thin layer of one of the recommended **VENUS** dentin shades first.
5. To avoid excess translucency and achieve an ideal esthetic result in the case of large Class IV cavi-

ties and anterior tooth reconstruction, it is necessary to utilize a **VENUS** dentin shade in the deepest layers, which can then be covered by the enamel and incisal **VENUS** shades.

6. Polymerise **VENUS** in thin layers (max. 2 mm) according to the table below using an acceptable QHT (Quartz-Halogen-Tungsten) curing light such as the Heraeus Kulzer Translux Curing unit or any unit with a similar spectrum and light-output ($> 400 \text{ mW/cm}^2$) During polymerisation, an oxygen inhibited-layer forms on the surface which should not be touched, contaminated or removed during the build-up phase of the restoration. It serves as chemical connection for any subsequent composite layers that are added.

| | | |
|--|-----------------------------------|------|
| A1, A2, A3, A3.5, B1, B2, C2, D2, T1, T2, T3, SB1, SB2, HKA2.5 | Curing time with QHT curing light | 20 s |
| A4, B3, C3, C4, D3, OA2, OA3, OA3.5, OB2, OC3, OD2, SBO, HKA5 | Curing time with QHT curing light | 40 s |
| All shades | Maximum layer thickness | 2 mm |

Recommended **VENUS** curing times with Translux Curing Unit or other QHT light curing units: For Plasma Arc light curing units, the curing time can usually be reduced by 25%.

For LED (light emitting diode) curing units, use the same curing times recommended with a QHT light if the light source can be held in close approximation to the composite material. However, due to the

relatively low light output of currently available LED curing units, it is recommended to increase curing time to at least 150%, if the light source must be positioned away from the surface of the material. Note: Curing lights vary greatly in intensity and spectral output. All curing times are minimum times. User should check with the light unit manufacturer and verify its efficacy non-clinically before using in a patient.

7. After final build-up and polymerisation is completed, **VENUS** may immediately be finished and polished. For gross finishing, use diamond burs. Fine finishing and shaping is best accomplished with fine-grained diamond stones and multifluted carbide burs. Proximal surfaces may be finished with abrasive strips. A high final polish can be achieved with flexible, silicone rubber polishing points, cups, discs, or similar. Composite polishing pastes can be applied with polishing brushes and small buffing wheels if desired.

Further recommendations and precautions:

- When building up large restorations or those, which require lengthy time to complete, the overhead operating light should temporarily be turned off or aimed away from the working area in order to prevent premature curing of the composite.
- The light curing times mentioned above are the minimum times recommended and should not be reduced in order to achieve optimal restoration quality and physical properties of the material.
- The PLT's (pre-loaded tips) are intended for single use only and cannot be resterilized or disinfected without damaging and contaminating the restorative material. To avoid cross contamination between patients, discard PLT after each single patient use.
- In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentin or the exposure locally with a calcium hydroxide liner covered with a glass iono-

mer cement before restoring with **VENUS**.

- Do not use eugenol containing materials in or around the cavity preparation being restored, because these may interfere with the proper curing of **VENUS**.
- Avoid contact with the skin! In case of contact with eyes rinse with plenty of water and seek medical advice if necessary.
- Do not use the material after the expiration date. Do not store above 25°C. Keep out of direct sunlight.
- To avoid waste, after the material has been applied from the syringe, relieve the internal pressure by unscrewing the plunger a quarter turn.
- Do not use **VENUS** in patients, which are allergic against the ingredients of **VENUS**.

Available Colors:

| Dentin | Enamel | Incisal |
|-------------------------------------|--|------------|
| OA2, OA3, OA3.5, OB2, OC3, OD2, SBO | A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, C4, D2, D3, SB1, SB2, HKA2.5, HKA5 | T1, T2, T3 |

Disclaimer:

Our technical advice, suggestions and recommendations for the use of this product is given in good faith and is based on our own tests and the best available current knowledge. But it is not to be construed as a warranty for use. This also applies where other manufacturer products have been referred to or the proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as and to determine their final suitability for their intended

purpose. The final application, use and processing of these products are beyond our control, and therefore, entirely the responsibility of the user. In the event that it can be shown that our product was delivered to you in a defective or damaged condition, our liability is limited to the replacement value of the goods delivered by us and used by you. We will, of course, provide consistent high quality products within the scope of our quality control standards and General Conditions of Sale and Delivery.

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Heraeus Kulzer GmbH
Heraeus Kulzer, Inc.